

Medical Carriers User Guide

Tested to maintain temperature of 2-8 degrees



By Appointment to
His Majesty The King
Supplier of Tamper Evident
Bags and Security Seals
Versapak International
Limited, Kent

Product Name/Code	Carrier	Thermal Barriers	Freeze Boards	Ambient Temperature	*Temperature maintained for
Small Pathology Sample Carrier (PYTB1)		x1 Small (THBARRIER_SML)	400g x3 (FREEZEB400)		
Small Vaccine Carrier (PYTBVAC1)				21°C 6hrs 53min 30°C 4hrs 07min	6hrs 53min 4hrs 07min
Medium Pathology Sample Carrier (PYTB2)		x1 Large (THBARRIER_LRG)	750g x3 (FREEZEB750)		
Medium Vaccine Carrier (PYTBVAC2)				21°C 22hrs 11min 30°C 9hrs 41min	22hrs 11min 9hrs 41min
Large Pathology Sample Carrier (PYTB3)		x2 Small (THBARRIER_SML)	750g x5 (FREEZEB750)		
Large Vaccine Carrier (PYTBVAC3)				21°C >24 hours 30°C 12hrs 39min	>24 hours 12hrs 39min

*Study by TuV GmbH. Freeze Boards preconditioned to -18°C for 24 hours. Thermal Barrier preconditioned to 4 degrees. For more information please contact us.


PLACEMENT



Place contents inside



Place Thermal Barriers

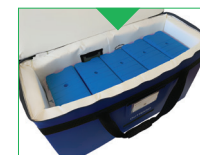

Place Freeze Boards



Close and Seal

