

Today's Training Session:




Understanding Sutures

What a minefield!

Suture Material Type

Absorbable

Non-Absorbable

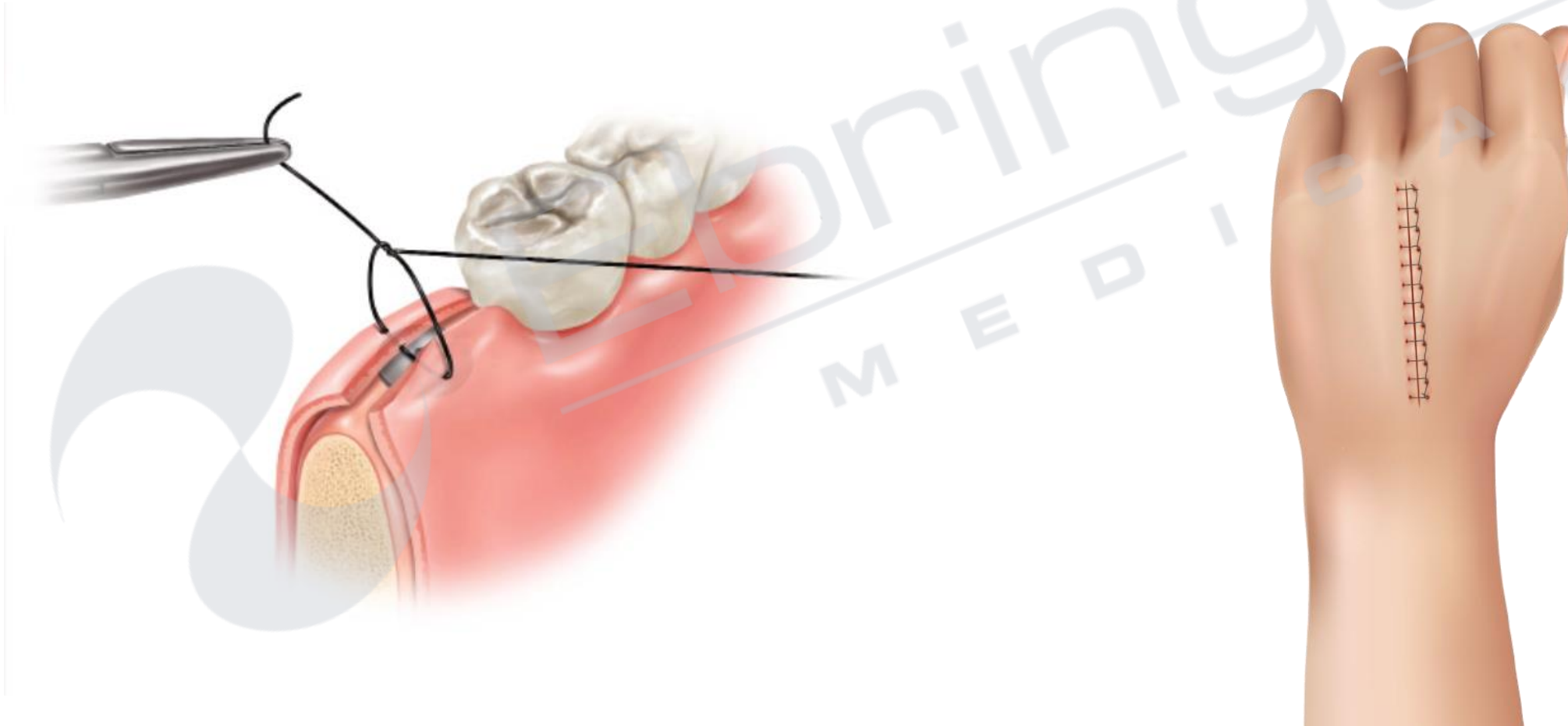

3-0
(2 Ph. Eur.)



27" (70 cm)

ETHICON®
Coated VICRYL™

Polyglactin 910

undyed braided **absorbable suture**

incolore tressé suture résorbable

ungefärbt geflochten chirurg. Nahtmaterial

resorbierbar

non colorata intrecciata sutura assorbibile

incolora trenzada sutura absorbible

incolor entrançada sutura absorvível

ongekleurd gevlochten chirur. hechtmateriaal resorbeerbaar

 **ETHICON, LLC**

STERILE

EO



2797

REF J416



GTIN:(01)10705031039667

CCN: XJJ416.P30

LOT SGMDRJ  2027-05-31



Suture Structure


Monofilament



Braided



Stratafix

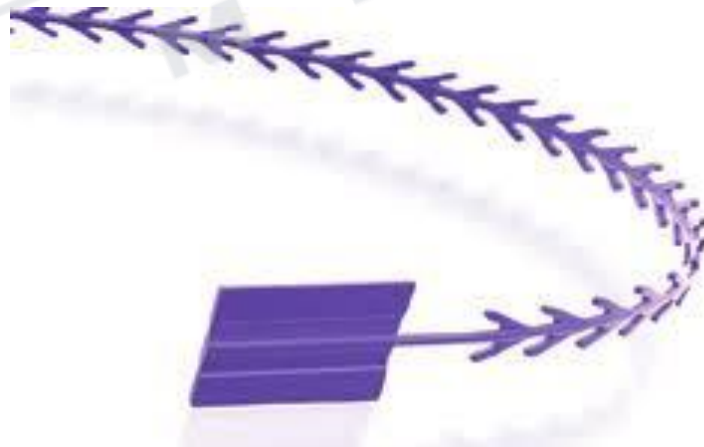

3-0
(2 Ph. Eur.)



27" (70 cm)

ETHICON®
Coated VICRYL™

Polyglactin 910

undyed braided absorbable suture

incolor tressé suture résorbable

ungefärbt geflochten chirurg. Nahtmaterial

resorbierbar

non colorata intrecciata sutura assorbibile

incolora trenzada sutura absorbible


incolor entrançada sutura absorvível

ongekleurd gevlochten chirur. hechtmateriaal resorbeerbaar

 **ETHICON, LLC**

STERILE

EO

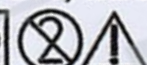

2797

REF J416



GTIN:(01)10705031039667

CCN: XJJ416.P30

LOT SGMDRJ  2027-05-31



Suture Gauge (Thread Diameter)

Suture USP

3

2

1

0

2-0

3-0

4-0

3-0
(2 Ph. Eur.)




27" (70 cm)

ETHICON®
Coated VICRYL™

Polyglactin 910
undyed braided absorbable suture
incolore tressé suture résorbable
ungefärbt geflochten chirurg. Nahtmaterial
resorbierbar
non colorata intrecciata sutura assorbibile
incolore trenzada sutura absorbibile
incolor entrançada sutura absorvível
ongekleurd gevlochten chirur. hechtmateriaal resorbeerbaar

 ETHICON, LLC

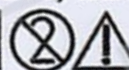STERILE **EO**   **C** **€** **2797**

REF J416




GTIN:(01)10705031039667

CCN: XJJ416.P30

LOT **SGMDRJ**  **2027-05-31**



Suture Length

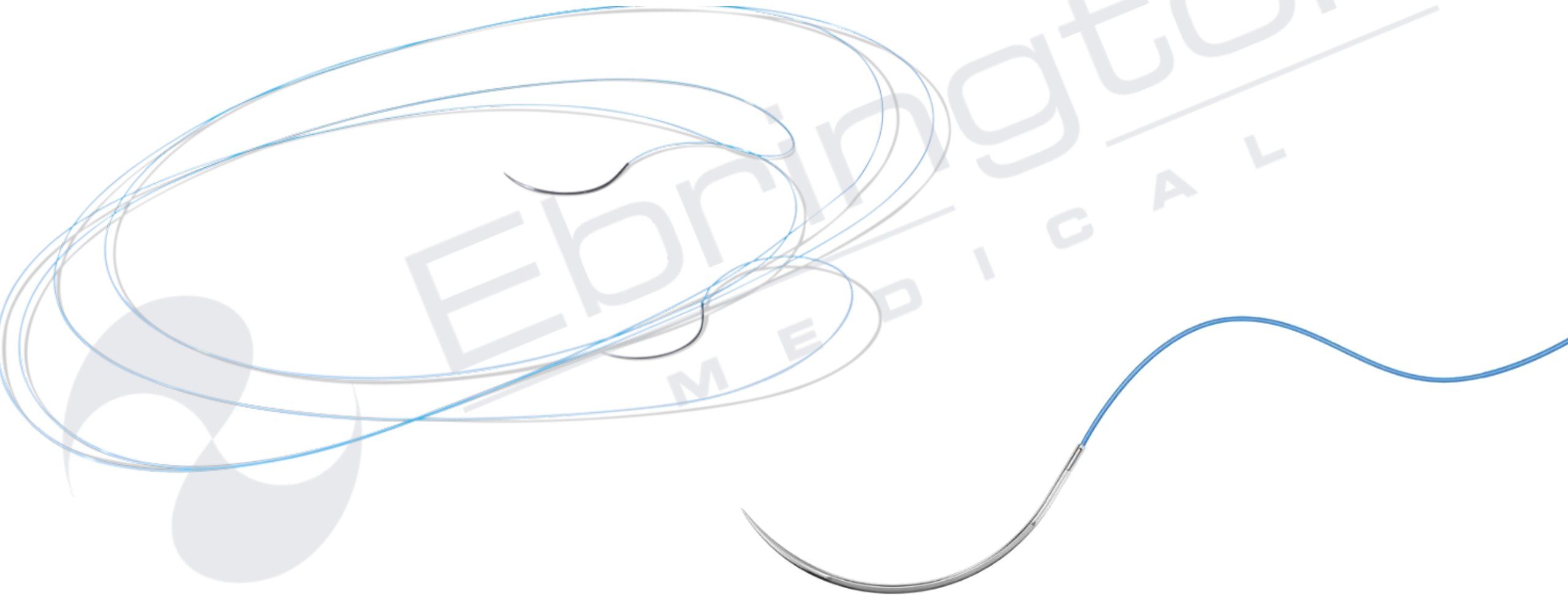

3-0
(2 Ph. Eur.)



27" (70 cm)

ETHICON®
Coated VICRYL™

Polyglactin 910

undyed braided absorbable suture

incolore tressé suture résorbable

ungefärbt geflochten chirurg. Nahtmaterial

resorbierbar

non colorata intrecciata sutura assorbibile

incolora trenzada sutura absorbible


incolor entrançada sutura absorvível

ongekleurd gevlochten chirur. hechtmateriaal resorbeerbaar

 **ETHICON, LLC**

STERILE

EO

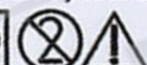

2797

REF J416



GTIN:(01)10705031039667

CCN: XJJ416.P30


 2027-05-31

LOT SGMDRJ



The Needle – Needle Shapes


1/4 Circle

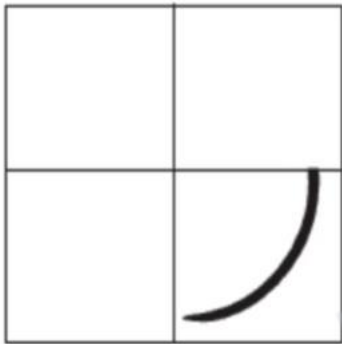


3/8 Circle



1/2 Circle

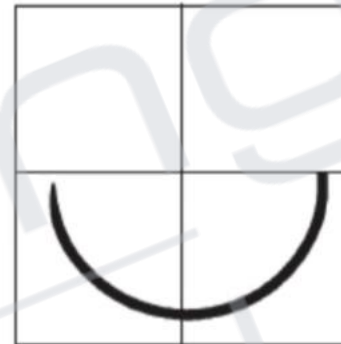


5/8 Circle

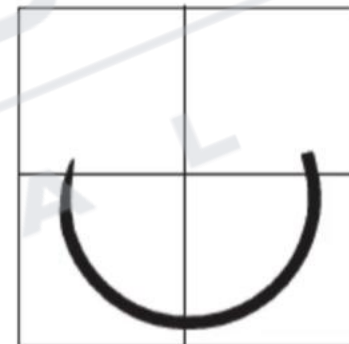


J Shape




Compound Curve

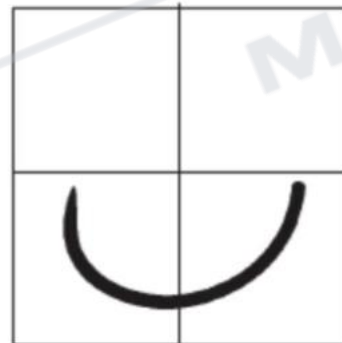

Straight

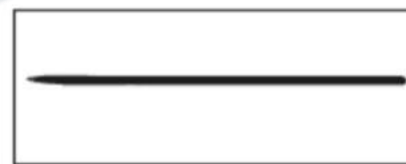

3-0
(2 Ph. Eur.)

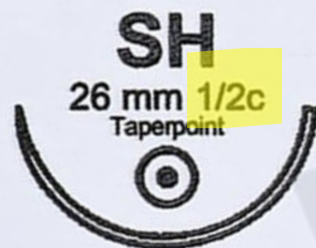

27" (70 cm)

ETHICON®
Coated VICRYL™

Polyglactin 910

undyed braided absorbable suture

incolore tressé suture résorbable

ungefärbt geflochten chirurg. Nahtmaterial

resorbierbar

non colorata intrecciata sutura assorbibile

incolora trenzada sutura absorbible


incolor entrançada sutura absorvível

ongekleurd gevlochten chirur. hechtmateriaal resorbeerbaar

 **ETHICON, LLC**

STERILE

EO

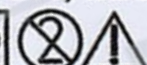

C **€** **2797**

REF J416




GTIN:(01)10705031039667

CCN: XJJ416.P30

LOT **SGMDRJ**  **2027-05-31**




The Needle – Needle Point

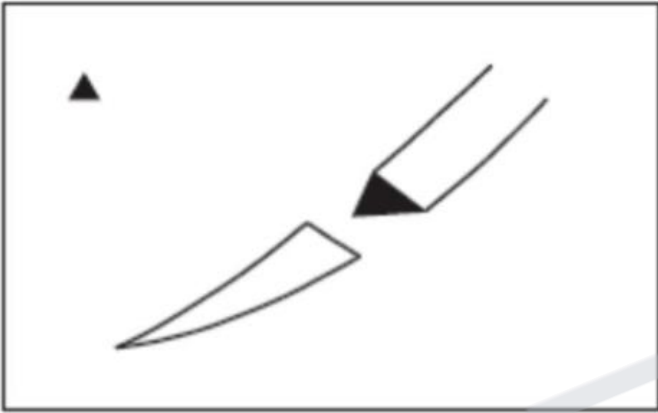


Conventional Cutting



PRIME Needle
Conventional Cutting

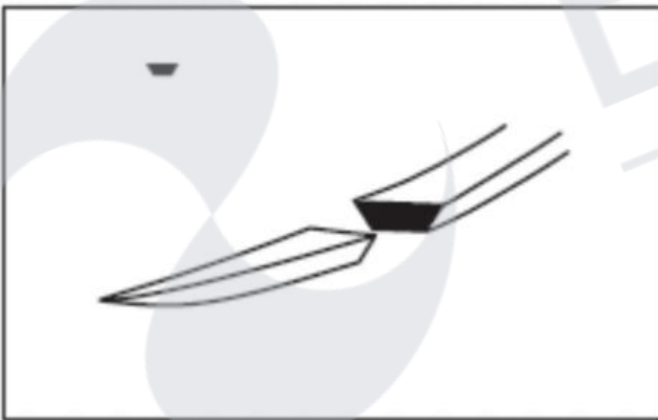

Spatula

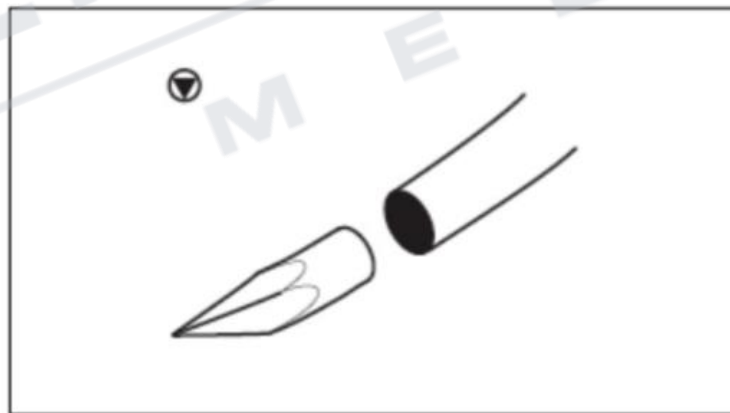

TAPERCUT Needle



3-0
(2 Ph. Eur.)



27" (70 cm)

ETHICON®
Coated VICRYL™

Polyglactin 910

undyed braided absorbable suture

incolore tressé suture résorbable

ungefärbt geflochten chirurg. Nahtmaterial

resorbierbar

non colorata intrecciata sutura assorbibile

incolora trenzada sutura absorbible


incolor entrançada sutura absorvível

ongekleurd gevlochten chirur. hechtmateriaal resorbeerbaar

 **ETHICON, LLC**

STERILE

EO

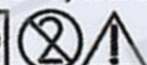

2797

REF J416



GTIN:(01)10705031039667

CCN: XJJ416.P30

 **2027-05-31**

LOT **SGMDRJ**




Size

CUTTING EDGE



TAPER POINT

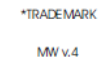

X mm

3-0
(2 Ph. Eur.)



27" (70 cm)

ETHICON®
Coated VICRYL™

Polyglactin 910

undyed braided absorbable suture

incolore tressé suture résorbable

ungefärbt geflochten chirurg. Nahtmaterial

resorbierbar

non colorata intrecciata sutura assorbibile

incolora trenzada sutura absorbible

incolor entrançada sutura absorvível

ongekleurd gevlochten chirur. hechtmateriaal resorbeerbaar

 **ETHICON, LLC**

STERILE **EO**     **2797**

REF J416




GTIN:(01)10705031039667

CCN: XJJ416.P30

LOT **SGMDRJ**  **2027-05-31**



The Needle – Needle Combination Codes

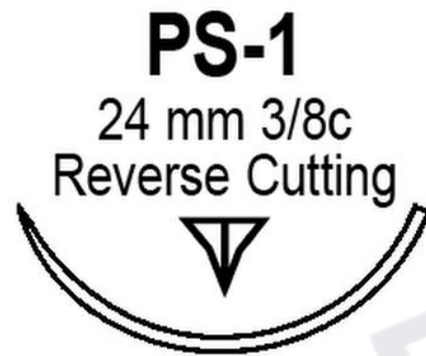

3-0
(2 Ph. Eur.)



27" (70 cm)

ETHICON®
Coated VICRYL™

Polyglactin 910
undyed braided absorbable suture
incolore tressé suture résorbable
ungefärbt geflochten chirurg. Nahtmaterial
resorbierbar
non colorata intrecciata sutura assorbibile
incolore trenzada sutura absorbibile
incolor entrançada sutura absorvível
ongekleurd gevlochten chirur. hechtmateriaal resorbeerbaar

 **ETHICON, LLC**

STERILE **EO**     **2797**

REF J416



GTIN:(01)10705031039667

CCN: XJJ416.P30

LOT **SGMDRJ**  **2027-05-31**



The Needle – Needle Uses

Needle Code	Needle Meaning
BB	Blue Baby
BV	Blood Vessel
C	Cardiovascular
CP	Cutting Point
FS	For Skin
M	Muscle
PC	Precision Cosmetic
SC	Straight Cutting
TN	Trochar Needle
X	Exodental

**Needle
Code**

**Needle
Meaning**

PS

Plastic Surgery

U

Urology

RD

Renal Detachment

CE

Cutting Edge

3-0
(2 Ph. Eur.)



27" (70 cm)

ETHICON®
Coated VICRYL™

Polyglactin 910

undyed braided absorbable suture

incolore tressé suture résorbable

ungefärbt geflochten chirurg. Nahtmaterial

resorbierbar

non colorata intrecciata sutura assorbibile

incolora trenzada sutura absorbible


incolor entrançada sutura absorvível

ongekleurd gevlochten chirur. hechtmateriaal resorbeerbaar

 **ETHICON, LLC**

STERILE

EO

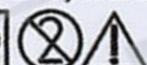

2797

REF J416



GTIN:(01)10705031039667





CCN: XJJ416.P30

LOT SGMDRJ  2027-05-31





J&J Suture Types

Absorbable

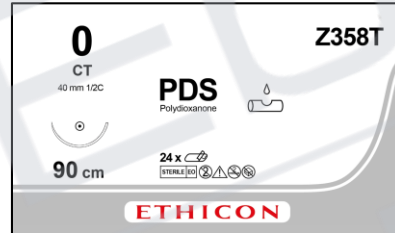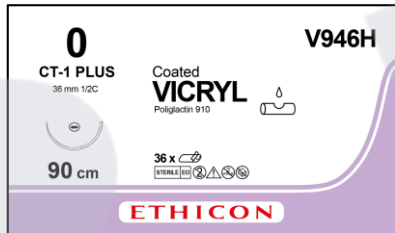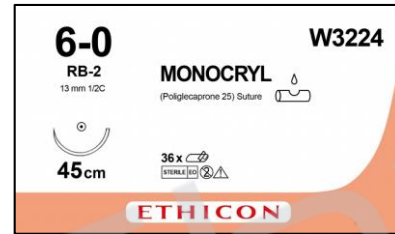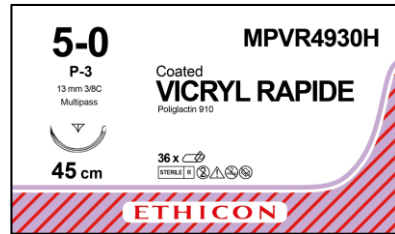


Non-Absorbable

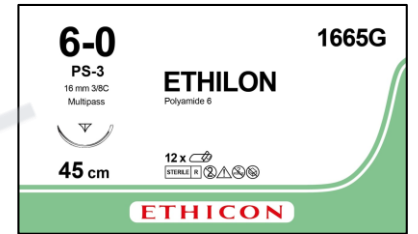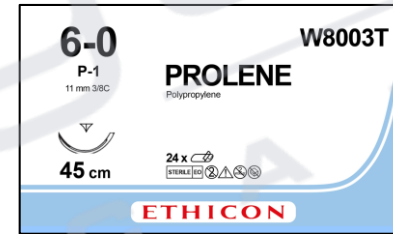


Antibacterial Sutures

Ethicon Plus Suture Range

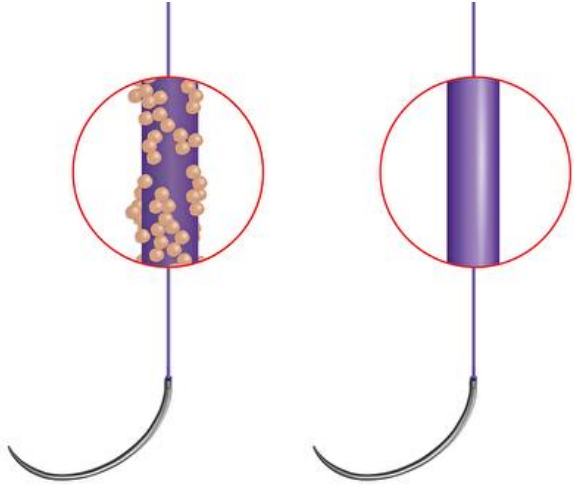


VICRYL Plus and another suture
in *Staphylococcus aureus*

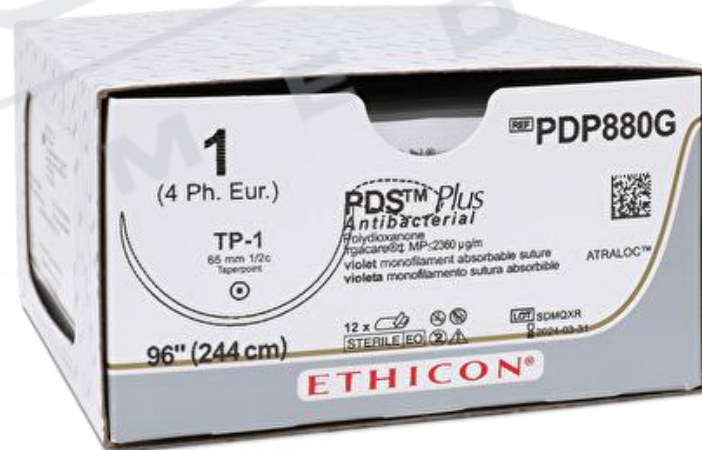

Stratafix Sutures

Knotless Tissue Control Device

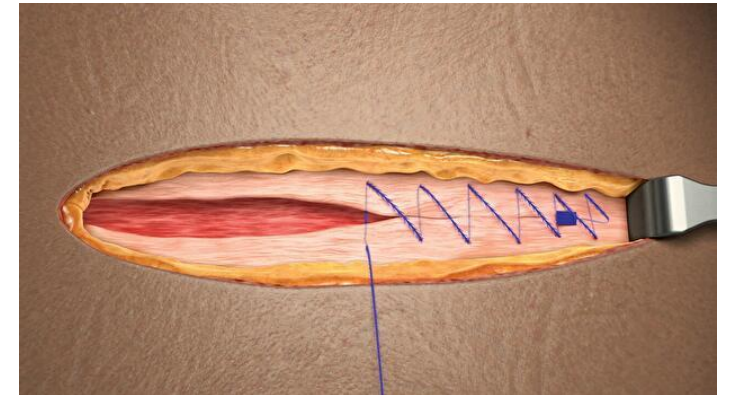

Select Sutures Online

www.eb-med.com - Try our product filter!

FREE Shipping Available - Click To View!

£ GBP

Sign In

Basket / £0.00

Ebrington
MEDICAL

Telephone: 028 7134 5188

Search here... 

ConsumablesEquipmentFurnitureTeaching AidsServices

Home / Consumables / Wound Closure / Sutures

Sutures



Looking for a specific suture?

Filter between the various types of materials, needles and brands to find the suture best suited for the procedure you have in mind.

As a Johnson & Johnson distributor we have access to their full wound closure product portfolio, please get in touch if you require a specific product not shown.

[Learn This Tip To Make Re-ordering Sutures Easy!](#)

ETHICON
Johnson & Johnson SURGICAL TECHNOLOGIES

Filter By Category

Adhesives, Strips and Staples

Sutures

Filter By

Absorbable

☐ Absorbable (167)

☐ Non-Absorbable (87)

Brand

☐ Ethibond Excel (7)

☐ Ethilon (23)

Popularity



Prolene Sutures W8020T x 24

Code: W8020T

£93.00 Ex VAT
(£111.60 Inc VAT)

Add To Basket



Ethilon Sutures 662G x 12

Code: 662G

£28.10 Ex VAT
(£33.72 Inc VAT)

Add To Basket



Vicryl Sutures W9437 x 12

Code: W9437

£58.50 Ex VAT
(£70.20 Inc VAT)

Add To Basket

Absorbable

☐ Absorbable (167)

☐ Non-Absorbable (87)

Colour

☐ Black (48)

☐ Green (5)

☐ White (3)

☐ Blue (33)

☐ Undyed (92)

☐ Violet (72)

Qty

☐ 6 (1)

☐ 12 (121)

☐ 24 (30)

☐ 36 (101)

Brand

☐ Ethibond Excel (7)

☐ Ethilon (23)

☐ Mersilene (1)

☐ Mersilk (26)

☐ Monocryl (19)

☐ Monocryl Plus (20)

☐ PDS II (7)

☐ PDS Plus (20)

☐ Prolene (34)

Gauge

☐ 0 (9)

☐ 1 (18)

☐ 2 (1)

☐ 2-0 (42)

☐ 3-0 (65)

☐ 4-0 (47)

☐ 5 (1)

☐ 5-0 (40)

☐ 5mm (1)

Suture Length

☐ 10cm (1)

☐ 20cm (1)

☐ 30cm (7)

☐ 45cm (97)

☐ 50cm (1)

☐ 60cm (7)

☐ 70cm (60)

☐ 75cm (48)

☐ 90cm (18)

Needle Code

☐ ASH-35 (1)

☐ BB (3)

☐ BV (1)

☐ BV-1 (3)

☐ C-1 (1)

☐ CC-1 (1)

☐ CC-20 (1)

☐ CC-40 (1)

☐ CC175-8 MP (1)

Needle Type

☐ 5/8 Circle Colateral Ligament (1)

☐ Aortic Half Circle (1)

☐ Blood Vessel (4)

☐ Blue Baby (3)

☐ Calcified Cornary (4)

☐ Cardiovascular (1)

☐ Circle Taper (4)

☐ Circle Taper Blunt (1)

☐ Circle Taper Extra Large (5)

Suture Structure

☐ Braided (129)

☐ Monofilament (120)

☐ Stratafix, Monofilament (5)

Needle Point

☐ 0 (8)

☐ Conventional Cutting (4)

☐ Ethiguard Blunt Taperpoint (1)

☐ Micro-Point Reverse Spatula (6)

☐ PC Prime Conventional Cutting (32)

☐ Prime Reverse Cutting (60)

☐ Reverse Cutting (54)

☐ Sabreloc Reverse Spatula (4)

Needle Length

☐ 6.5cm (1)

☐ 6.5mm (5)

☐ 8mm (4)

☐ 9.3mm (4)

☐ 11mm (9)

☐ 13mm (18)

☐ 16mm (36)

☐ 17mm (15)

☐ 19mm (36)

Needle Shape

☐ 0 (8)

☐ 1/2 Circle (81)

☐ 1/4 Circle (1)

☐ 3/8 Circle (147)

☐ 5/8 Circle (1)

☐ Comp Curve (2)

☐ Fish Hook (2)

☐ Ski (1)

☐ Straight (9)



The End
Thank You For Your Time
Any Questions?